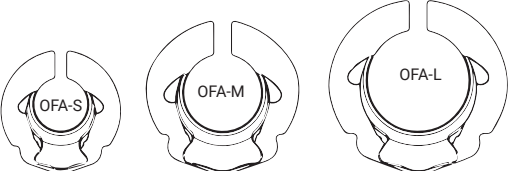
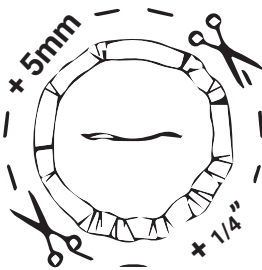
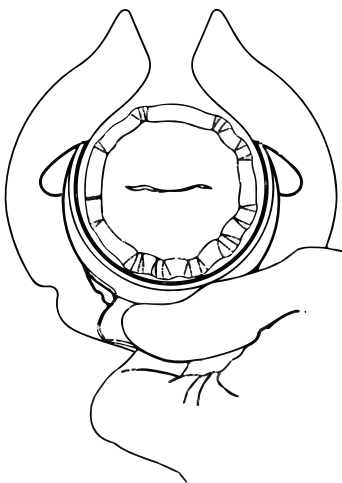
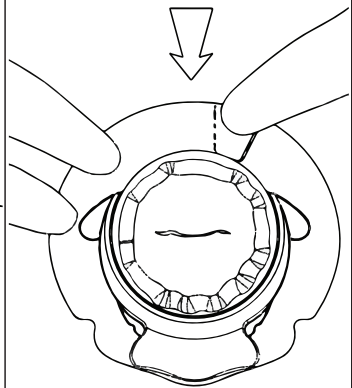
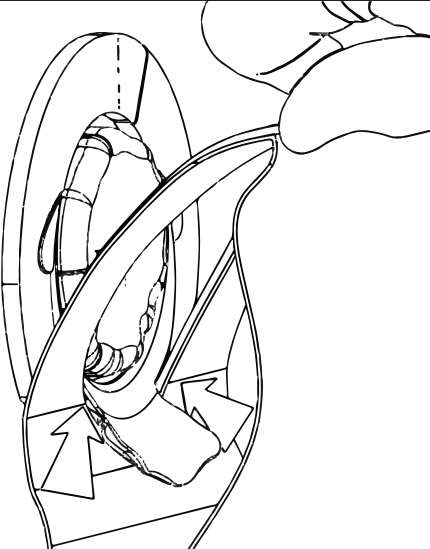
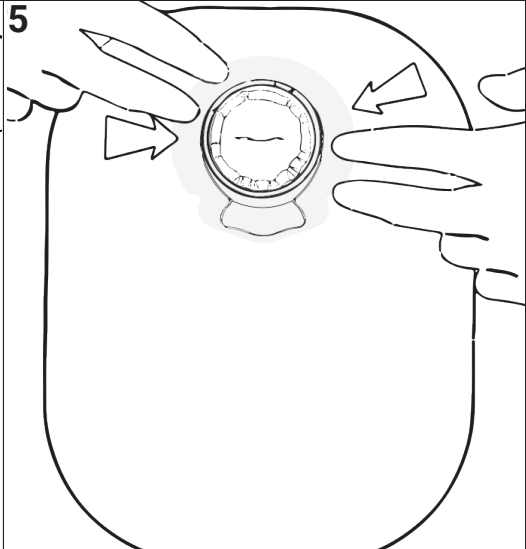


Ostoform Moldable Seal with FLOWASSIST Protection

**1****2****3****4****5**

Intended Use: Non-sterile, single-use Medical Device that aims to manage and prevent skin complications for ostomy pouch users.

Product Features: Moldable Ostomy Seal with a non-absorbent spout, which directs stoma output away from your skin.

Application:

1. Cut the pouch opening to stoma size + 5mm ($1/4$ "
e.g. for a 25mm stoma, cut the pouch opening to 30mm
e.g. for a 1" stoma, cut the pouch opening to ($1\ 1/4$ ")
2. Position the spout directly below the stoma.
3. Wrap the tabs around the stoma to ensure a snug fit.
4. For secure pouch attachment, ensure the spout is fully inside the pouch opening, then pull the pouch up into the slot underneath the spout.
5. Press the pouch adhesive against the Seal and skin to achieve good adhesion.

Note: If you use a 2-piece pouching system, position the baseplate over the Seal first. Then secure the 2-piece pouch to the baseplate.

⚠ Warnings and Precautions:

1. Ensure the spout does not block the stoma.
2. The device is single-use. Dispose after use.
3. The Ostoform Seal has passed biocompatibility testing. If you feel any irritation or discomfort, please discontinue using the device and consult your ostomy specialist.
4. Should a serious incident arise as a result of using this device please report this immediately to the manufacturer and the local Competent Authority.

Storage: Store below 30°C / 86°F in dry conditions and out of direct sunlight.

Shelf Life: Please refer to the expiry date (📅) on device label.

Warranty: The product will function as intended up to the expiry date.

Product Composition:

Moldable Seal: Integrated hydrocolloid adhesive (non-recyclable)
FLOWASSIST Protection: Thermoplastic Elastomer (non-recyclable)

Disposal: Dispose of in a safe and approved manner in accordance with local regulations for disposing of non-recyclable materials.
Do not flush down the toilet.

IFU Ref: 01-IFU-005 2020-05-06



Ostoform Limited,

2nd Floor, Block A, Marlinstown Business Park,
Mullingar, Co. Westmeath, N91 WR96, Ireland
E: info@ostofrom.com www.ostofrom.com

