**Intended Use:** A single-use medical device (system) that acts as a mechanical barrier to intact and/or injured peristomal skin; directs output to and collects output in a pouch for people who have a stoma with fluidic output.

**Product Features:** Integrated, laminated hydrocolloid baseplate with non-absorbent FLOWASSIST spout (A), which directs stoma output away from the skin and into a pouch.

**Clinical Benefit:** Assist in managing stoma output collection, appliance leaks and peristomal skin complications.

**Preparation**

1. Wash your hands and ensure the skin around the stoma is clean and dry. Shave any hair around the stoma if necessary.
2. Using scissors, cut the baseplate opening to match the size of the stoma. Refer to the stoma size guide on the back of the baseplate.

**⚠ WARNING:** Do not cut the FLOWASSIST Spout (A).

**Applying the Baseplate and Pouch**

3. Warm the baseplate by holding it in your hands for 30 seconds.
4. Remove the plastic liner.
5. Position the FLOWASSIST Spout below the stoma or rotate to align with the opening of the stoma. Apply the adhesive side of the baseplate to the skin around the stoma and gently press in place for 30 seconds.
6. Attach the pouch to the baseplate by placing your fingers underneath the floating flange (B) and pressing the two plastic click rings together. Start from the bottom and work your way around until the rings are fully clicked together.

**DRAINABLE**

- 7a. Close the Drainable Pouch by folding the drainable outlet (C) upward twice. Secure with hook and loop fastener (D).
- 8a. Empty the Drainable Pouch by releasing the hook and loop fastener, unfolding the outlet, and pinching the sides over a toilet.

**UROSTOMY**























- 7b. Close the Urostomy Pouch by placing the stopper (E) on the valve (F) and pressing firmly. Ensure the tap (G) is turned to the closed position.
- 8b. Empty the Urostomy Pouch by bringing the valve upwards to the level of the stoma, then releasing the stopper from the valve. Position over a toilet, and then twist tap to open valve.

**Changing The System**

9. Remove the pouch by holding the baseplate firmly against your abdomen with one hand and using the other hand to gently pry the pouch click ring away from the baseplate. You may attach a new pouch (Step 6) or change the whole system (Step 10).
10. Remove the baseplate by gently peeling it away from your abdomen. Use your second hand to support the skin adjacent to the baseplate as you peel. Use adhesive remover if desired.
11. Dispose of in a safe and approved manner in accordance with local regulations. Do not flush down the toilet.

**Optional Steps**

12. Filter stickers can be used with Drainable and Closed Pouches during activities where your pouch is going to be exposed to large amounts of water, e.g. Showering, swimming, etc.
13. Urostomy extension tubes can be fitted when you intend on using an overnight/overflow bag. For overnight use, attach an adapter to allow for connection with a urine collection pouch.

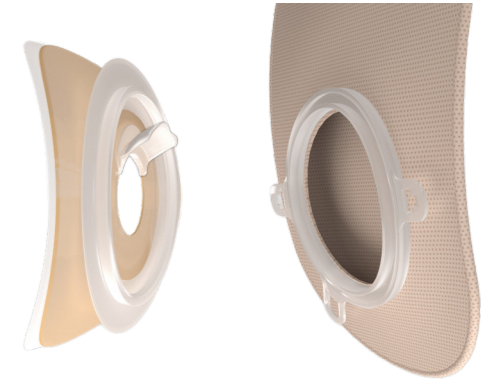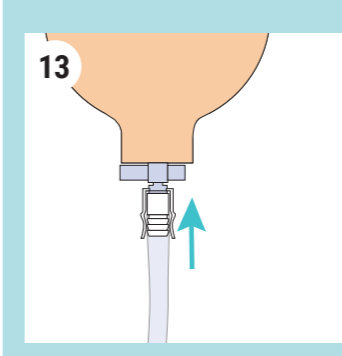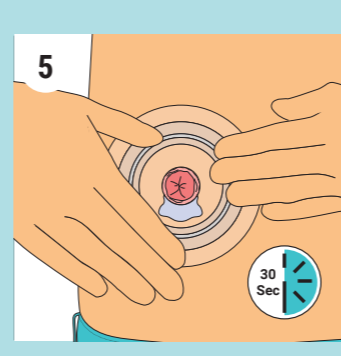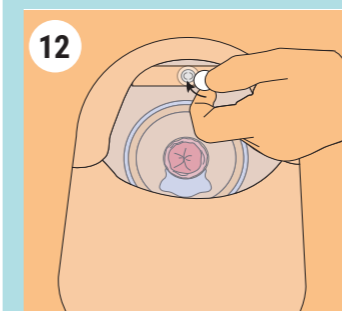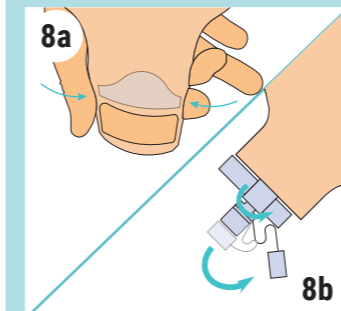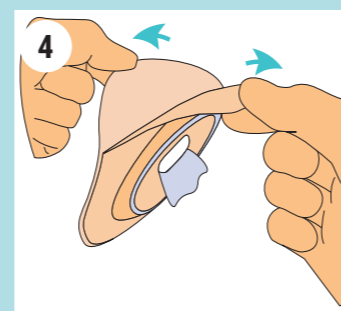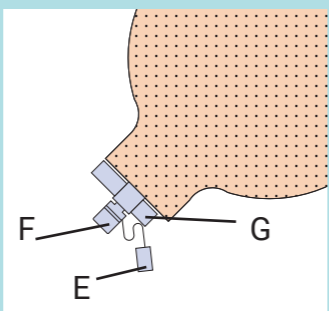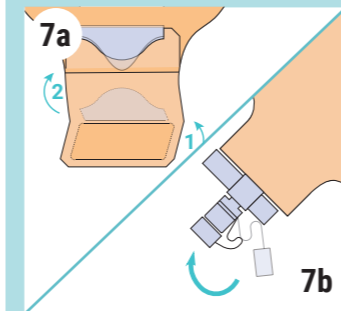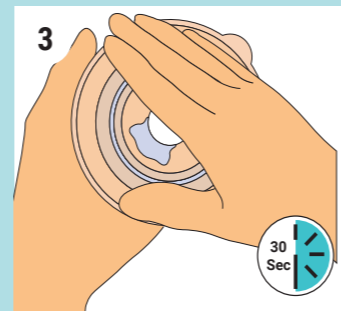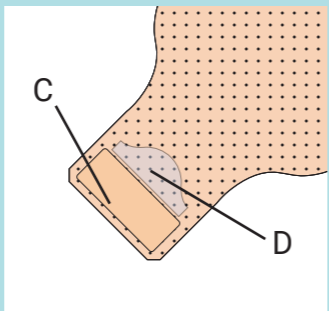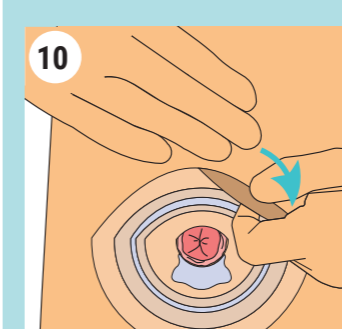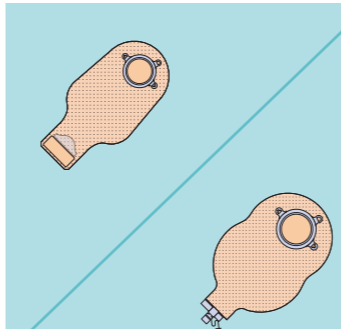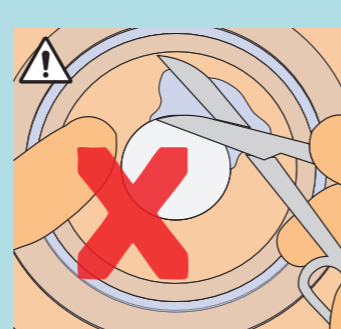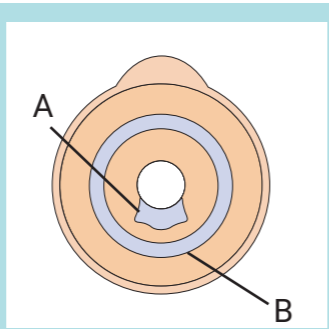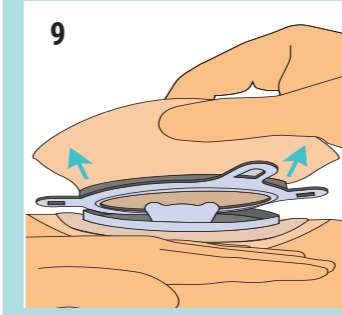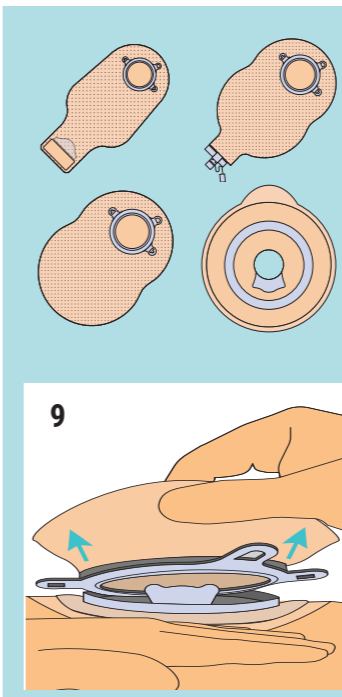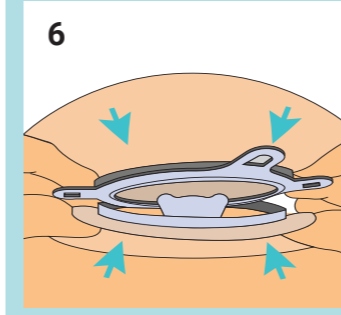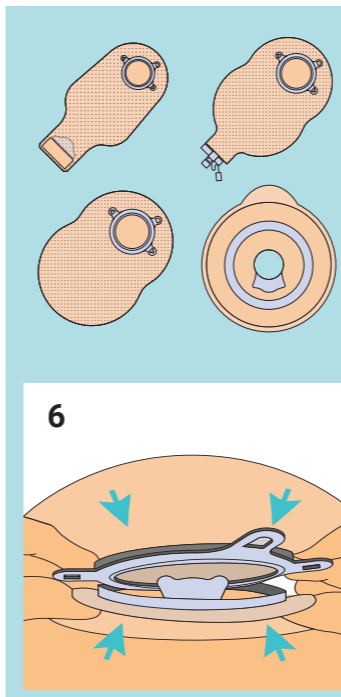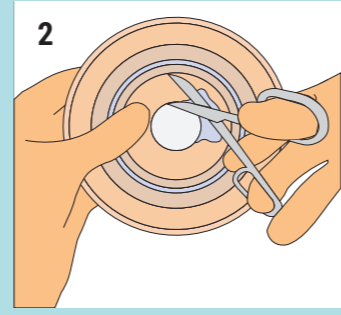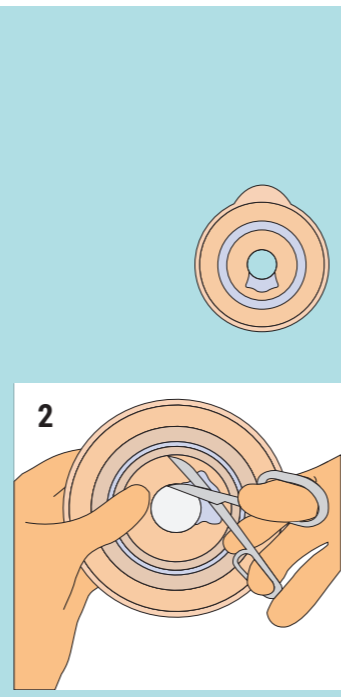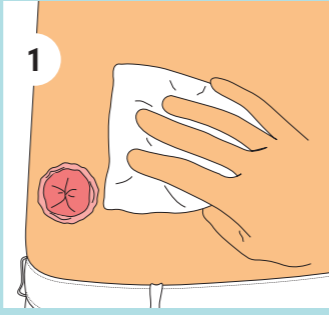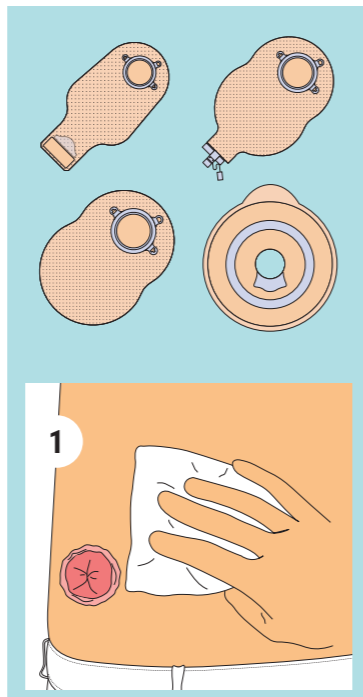**⚠ Warnings and Precautions:**

1. Ensure the spout does not block stoma output.
2. The device is single-use. Dispose after use. Re-use of this product may create risk to the user, such as loss of device functionality and/or infection.
3. If you feel any irritation or discomfort around the stoma, please discontinue using the device and consult an ostomy specialist.
4. Should a serious incident arise as a result of using this device, please report to Ostoform and the local Competent Authority.
5. The length of time you can use each baseplate and pouch is dependent on various environmental factors and should be closely monitored. Avoid filling the pouch over 2/3.
6. If the opening in the baseplate is cut or fit incorrectly, leakage may occur and may lead to skin damage.
7. Please seek advice from your healthcare professional before using convex devices.
8. Cannot be used simultaneously with the Ostoform Seal with FLOWASSIST Protection

**Storage:** Store at room temperature in dry conditions and out of direct sunlight.

**Shelf Life:** Please refer to the expiry date (📅) on device label.

**Warranty:** The product will function as intended up to the expiry date.

**Baseplate Product Composition:** Integrated, laminated hydrocolloid adhesive (non-recyclable) and thermoplastic elastomer (non-recyclable).

**Pouch Product Composition:** Odor-proof and low-noise, multi-layered film and thermoplastic elastomer (non-recyclable).

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