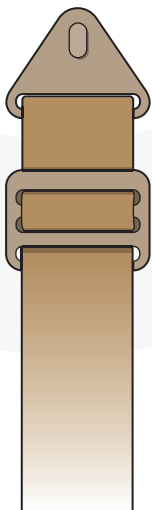
INSTRUCTIONS FOR USE
www.ostoform.com

www.ostoform.com
Info@ostoform.com

IFU Ref: 04A-IFU-001-US 2023-03-10

Manufactured in Germany for: Ostoform Limited,
 Unit 5C, Fairgreen House, Saint Loman's Terrace,
 Mullingar, Co. Westmeath N91 H635, Ireland.

	REF	
Ostoform Belt	OFA-OB-120	120cm

INTENDED USE

The Ostoform belt is an accessory product designed for individuals with an ostomy. It helps to secure the Ostoform pouching system to the body by providing an extra fixing point.

WARNINGS

1. Please only use the Ostoform belt according to the instructions provided and for its intended purpose.
2. Do not dispose of this product in the toilet. Proper disposal methods should be followed.
3. If you feel any irritation or discomfort around the stoma, please discontinue using the device and consult an ostomy specialist.
4. Should a serious incident arise as a result of using this device, please report to Ostoform and the local Competent Authority.

STORAGE

Store at room temperature in dry conditions and out of direct sunlight.

SHELF LIFE

Please refer to the expiry date on device label.

WARRANTY

The product will function as intended up to the expiry date.

BELT COMPOSITION

Viscose, rubber-coated elastane, and cotton.

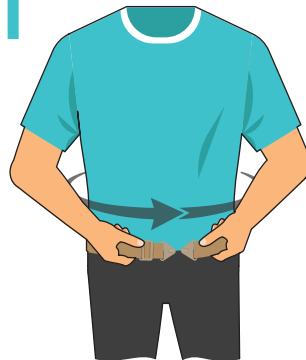
GLOSSARY

www.ostoform.com/symbol-glossary/


FITTING INSTRUCTIONS

1


Adjust the size of the belt to comfortably fit your waist.

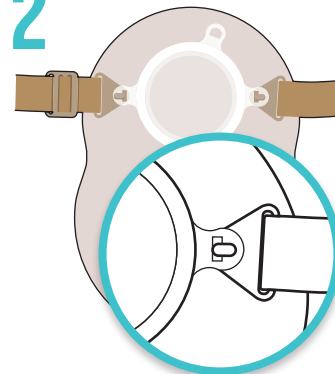2


Gently stretch the belt and fasten the hook to the eyelet on the system.

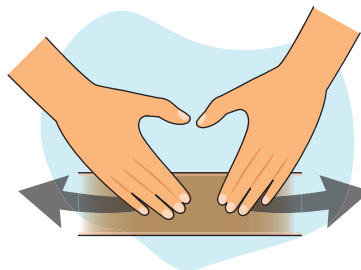3

Smooth out the belt, ensuring there are no twists.

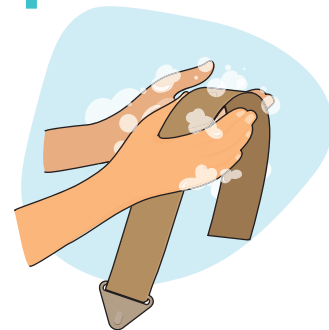4

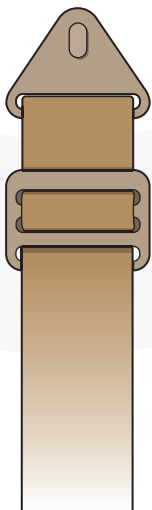
The ostomy belt should be cleaned after frequent use by hand washing with warm water (86°F/30°C).

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Fabricado en Alemania para: Ostoform Limited,