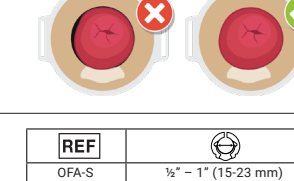
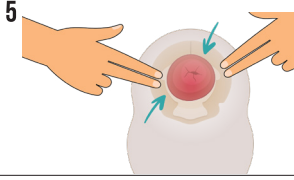
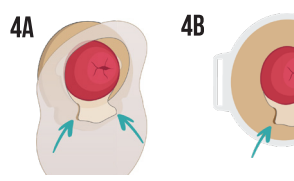
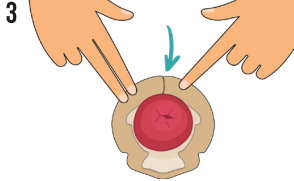
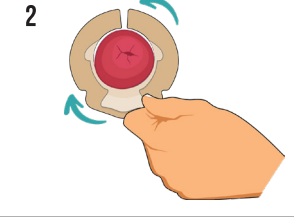
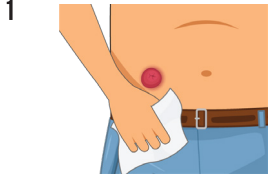
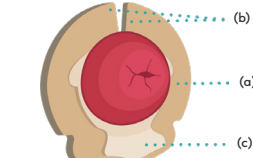
IFU REF: 03-IFU-007-EU 09/23 Rev.07

www.ostoform.com
email: info@ostoform.com

REF	Size
OFA-S	1/2" - 1" (15-23 mm)
OFA-M	1" - 1 1/4" (23-30 mm)
OFA-L	1 1/4" - 1 1/2" (30-40 mm)

Ostoform Limited, Unit 5C, Fairgreen House, Saint Loman's Terrace, Mullingar, Co. Westmeath N91 H635, Ireland.

(EN) INTENDED USE
Non-sterile, single-use Medical Device that aims to manage and prevent skin complications for people with an Ostomy/Fistula.

PRODUCT FEATURES
Moldable Ostomy Seal with a non-absorbent FLOWASSIST spout, which directs stoma output away from your skin.

CLINICAL BENEFIT
Assist in managing stoma output collection, appliance leaks and peristomal skin complications.

- APPLICATION**
1. Wash your hands and ensure the skin around your stoma is clean and dry before fitting the FLOWASSIST Seal. Remove the Seal from its packaging. Warm the Seal in your hands for 30 seconds and remove the clear liners.
 2. Position the FLOWASSIST spout (c) below the stoma or rotate to align with the opening of the stoma. You can hold the spout to help position the Seal.
 3. Wrap the ends (b) of the Seal adhesive (a) around the stoma ensuring the ends overlap.
 4. Prepare the pouch opening as per manufacturer guidelines. Ensure the FLOWASSIST spout is fully inside the pouch opening, then pull the 1-piece pouch (see figure 4A) or the 2-piece pouching system (see figure 4B) up underneath the spout, as far as it will go.
 5. Apply light pressure to the pouch to achieve secure adhesion.

Note: Visual inspection (See figure 6): If possible, check that no part of the Seal adhesive is visible. You should only see the FLOWASSIST spout (c). If you notice that an area of the Seal adhesive is exposed, try to adjust the pouch to cover this area. If you need a larger size or where the skin is irregular, the Ostoform Crest can be used with the Seal to build up, or extend, as necessary.

- WARNINGS AND PRECAUTIONS**
1. Ensure the spout does not block the stoma output.
 2. The device is single use. Dispose after use. Reuse of this single use product may create risk to the user, such as poor adhesion and/or infection.
 3. The FLOWASSIST Seal has passed biocompatibility testing. If you experience any irritation, discomfort or blockage, please discontinue using the device and consult your ostomy specialist.
 4. Should a serious incident arise as a result of using this device, please report to the manufacturer and the local Competent Authority.
 5. Closely monitor the wear time, as product performance depends on various environmental factors and break down can be an expected behavior. If breakdown occurs, replace with a new FLOWASSIST Seal.
 6. Incorrect fitting of the product may cause leakage to occur and may lead to skin damage.

STORAGE
Store between 0°C/32°F - 30°C/86°F in dry conditions and out of direct sunlight.

SHELF LIFE
Please refer to the expiry date (📅) on device label.

WARRANTY
The product will function as intended up to the expiry date.

PRODUCT COMPOSITION
Integrated hydrocolloid adhesive (non-recyclable) and thermoplastic elastomer (non-recyclable).

DISPOSAL
Dispose of in a safe and approved manner in accordance with local regulations. Do not flush down the toilet.

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