INSTRUCTIONS FOR USE

IFU REF 06-IFU-001

 www.ostoform.com
Info@ostoform.com

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Ostoform Limited,
Unit 5C,Fairgreen House,
Saint Loman's Terrace
Mullingar, Co. Westmeath,
N91H635 Ireland.

	REF	
Ostoform FLOWASSIST Flat Baseplate	OFA-FB57-S	¾ - 1" / (19-25 mm)
	OFA-FB57-M	1 – 1¼" / (25-32 mm)
	OFA-FB57-L	1¼" – 1⅞" / (32–40 mm)
Ostoform FLOWASSIST Convex Baseplate	OFA-CB57-S	¾ - 1" / (19-25 mm)
	OFA-CB57-M	1 – 1¼" / (25-32 mm)
	OFA-CB57-L	1¼" – 1⅞" / (32–40 mm)
Ostoform FLOWASSIST Drainable Pouch	OFA-DP57	-
Ostoform FLOWASSIST Urostomy Pouch	OFA-UP57	-
Ostoform FLOWASSIST Closed Pouch	OFA-CP57	-

- EN**
- Intended Use:** A single-use medical device (system) that acts as a mechanical barrier to intact and/or injured peristomal skin; directs output to and collects output in a pouch for people who have a stoma with fluidic output.
- Product Features:** Integrated, laminated hydrocolloid baseplate with non-absorbent FLOWASSIST spout (A), which directs stoma output away from the skin and into a pouch.
- Clinical Benefit:** Assist in managing stoma output collection, appliance leaks and peristomal skin complications.
- Preparation**
- Wash your hands and trim hair around the stoma if necessary. Ensure the skin around the stoma is clean and dry.
 - Using scissors, cut the baseplate opening to match the size of the stoma. Refer to the stoma size guide on the back of the baseplate.
- WARNING:** Do not cut the FLOWASSIST Spout (A).
- Applying the Baseplate and Pouch**
- Warm the baseplate by holding it in your hands for 30 seconds.
 - Remove the plastic liner.
 - Position the FLOWASSIST Spout below the stoma or rotate to align with the opening of the stoma. Apply the adhesive side of the baseplate to the skin around the stoma and gently press in place for 30 seconds.
 - Attach the pouch to the baseplate by placing your fingers underneath the floating flange (B) and pressing the two plastic click rings together. Work your way around until the rings are fully clicked together.
- Drainable**
- Close the Drainable Pouch by folding the drainable outlet (C) upward twice. Secure with hook and loop fastener (D).
 - Empty the Drainable Pouch by releasing the hook and loop fastener, unfolding the outlet, and pinching the sides over a toilet.
- Urostomy**
- Close the Urostomy Pouch by inserting the plug (E) into the drainage outlet.
 - To empty the Urostomy pouch position over a toilet and pull the plug (E) out of drainage outlet.
- Changing The System**
- Remove the pouch by holding the baseplate firmly against your abdomen with one hand and using the other hand to gently detach the pouch click ring away from the baseplate. You may attach a new pouch (Step 6) or change the whole system (Step 10).
 - Remove the baseplate by gently peeling it away from your abdomen. Use your second hand to support the skin adjacent to the baseplate as you peel. Use adhesive remover if desired.
 - Dispose of in a safe and approved manner in accordance with local regulations. Do not flush down the toilet.
- Optional Accessories (included with product)**
- Drainable and Closed:** Filter stickers can be used with Drainable and Closed Pouches during activities where your pouch is going to be exposed to large amounts of water, e.g. Showering, swimming, etc.
 - Urostomy:** Universal connectors can be fitted when you intend on using an overnight/ overflow bag.

- Warnings and Precautions:**
- Ensure the spout does not block stoma output.
 - The device is single-use and non-sterile. Dispose after use. Re-use of this product may create risk to the user, such as loss of device functionality and/or infection.
 - This product has passed biocompatibility testing. If you feel any irritation or discomfort around the stoma, please discontinue using the device and consult an ostomy specialist.
 - Should a serious incident arise as a result of using this device, please report to Ostoform and the local Competent Authority.
 - Closely monitor the wear time, as product performance depends on various environmental factors and breakdown can be an expected outcome. Replace the baseplate and pouch when necessary. Avoid over filling the pouch.
 - If the opening in the baseplate is cut or fit incorrectly, leakage may occur and may lead to skin damage.
 - Please seek advice from your healthcare professional before using convex devices.
 - If product is used on injured skin at site of adhesion, please use it with caution.
- Storage:** Store between (0°C/32°F - 30°C/86°F) in dry conditions and out of direct sunlight.
- Shelf Life:** Please refer to the expiry date (📅) on device label.
- Warranty:** The product will function as intended up to the expiry date.
- Baseplate Product Composition:** Integrated, laminated hydrocolloid adhesive (non-recyclable) and thermoplastic elastomer (non-recyclable) and latex free.
- Pouch Product Composition:** Odor-proof and low-noise, multi-layered film and thermoplastic elastomer (non-recyclable) and latex free.
- Glossary:** www.ostoform.com/symbol-glossary/

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