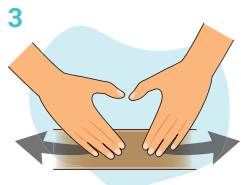
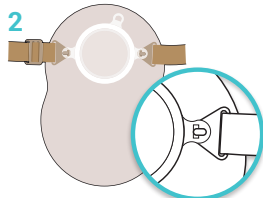
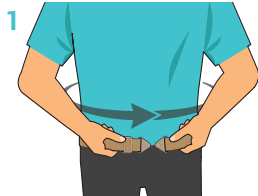
REF	
OFA-OB-120	120cm

IFU Ref: 06.1-IFU-001
2025-10-10

www.ostoform.com
 e:info@ostoform.com

Manufactured in Germany for/
Fabriqué en Allemagne pour/
Fabricado en Alemania para:
Ostoform Limited, Unit 5C,
Fairgreen House, Saint
Loman's Terrace, Mullingar,
Co. Westmeath, N91 H635,
Ireland.

Ostoform Limited, Unit 5C,
Fairgreen House, Saint
Loman's Terrace, Mullingar,
Co. Westmeath, N91 H635,
Ireland.

EN INSTRUCTIONS FOR USE

Intended use
The Ostoform Belt is an accessory product designed for individuals with an ostomy. It helps to secure the Ostoform pouching system to the body by providing an extra fixing point.

Fitting Instructions

1. Adjust the size of the belt to comfortably fit your waist.
2. Gently stretch the belt and fasten the hook to the eyelet on the system.
3. Smooth out the belt, ensuring there are no twists.
4. The ostomy belt should be cleaned after frequent use by hand washing with warm water (86°F/30°C).

Warnings

1. Please only use the Ostoform Belt according to the instructions provided and for its intended purpose.
2. Do not dispose of this product in the toilet. Proper disposal methods should be followed.
3. If you feel any irritation or discomfort around the stoma, please discontinue using the device and consult an ostomy specialist.
4. Should a serious incident arise as a result of using this device, please report to Ostoform and the local Competent Authority.

Storage: Store at room temperature in dry conditions and out of direct sunlight.

Shelf Life: Please refer to the expiry date (📅) on device label.

Warranty: The product will function as intended up to the expiry date.

Belt Composition: viscose, rubber-coated elastane, and cotton.

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